




MOVES® SLC™ Maintenance Schedule


Follow the below maintenance schedule to keep your MOVES® SLC™ device in optimal working condition.

Maintenance Frequency	Maintenance Procedure	Operator's Manual Procedure #	Maintenance Item	Maintenance Kit Part #
In-Use & During Long-Term Storage: Every 3 months prior to & post long-term storage	Exercising the Oxygen Concentrator	16.3.5	No supplies required	n/a
In-Use & During Long-Term Storage: Every 3 months prior to & post long-term storage	Battery Charging 	16.3.6	No supplies required	n/a
In-Use: Every 3 months prior to long-term storage	Removing / Replacing the Ventilator Filter 	16.3.2	Kit, MOVES® SLC™, Ventilator Filter, 10pk	125940
In-Use: Every 3 months prior to long-term storage	Removing / Replacing the Hydrocarbon Filter 	16.3.3	Kit, MOVES® SLC™, Hydrocarbon Filter, 6pk	124354

MOVES® SLC™ Maintenance Schedule

Follow the below maintenance schedule to keep your MOVES® SLC™ device in optimal working condition.

**THORNHILL
MEDICAL**
Inspiring Innovation

Maintenance Frequency	Maintenance Procedure	Operator's Manual Procedure #	Maintenance Item	Maintenance Kit Part #
In-Use: Every 3 months prior to long-term storage	Removing / Replacing the Ventilator Cartridge Liner 	16.3.4	Kit, MOVES® SLC™, Ventilator CartridgeLiner, 10pk	127407
Replace ventilator bag after each vent runtime of 2000 hours	Removal / Replacement of the Ventilator Bag	Refer to Service Manual	Kit, MOVES® SLC™ Ventilator bag	125943
<ul style="list-style-type: none"> In-Use & During Long- Term Storage: Every 12 months PRIOR TO & POST LONG- TERM STORAGE AFTER ANY REPAIR/REPLACEMENT PROCEDURES ARE COMPLETED 	Service Testing	Refer to Service Manual	n/a	n/a

We're Here to Help

Stay connected to MOVES® SLC™ and the Thornhill Medical Team.



Customer Care is one click away.
Access our Customer Care Portal
<https://thornhillmedical.com/customer-care/>



Training and Information in the Palm of your Hand
Download the Training App
<https://thornhillmedical.com/resources/moves-slc/videos/>



**THORNHILL
MEDICAL**
Inspiring Innovation

+1 416-597-1325 | 1-888-597-1325
60 Wingold Ave., Toronto, ON
M6B 1P5 Canada

info@thornhillmedical.com | www.thornhillmedical.com