

MADM™ Maintenance Schedule

Follow the below maintenance schedule to keep your MADM™ device in optimal working condition.

Maintenance Frequency	Maintenance Procedure	Service Manual Procedure #	Maintenance Item Description	Maintenance Kit Part #
<ul style="list-style-type: none"> • IN USE: Every 6 Months • PRIOR TO LONG-TERM STORAGE 	 Clearing or Removal/ Replacement of the Anesthetic Cartridge Fitting*	See Procedure # 6.20 for Complete Maintenance Steps	Kit, MADM, Anesthetic Cartridge Fitting	125928*
<ul style="list-style-type: none"> • IN USE: Every 6 Months • PRIOR TO LONG-TERM STORAGE 	 Removal/Replacement of the Purged Anesthetic Container	See Procedure # 6.7 for Complete Maintenance Steps	Kit, MADM, Purged Anesthetic Container	125915
<ul style="list-style-type: none"> • IN USE: Every 12 Months • PRIOR TO LONG-TERM STORAGE 	 Cleaning or Removal/ Replacement of the Heater*	See Procedure # 6.23 for Complete Maintenance Steps	Kit, MADM, Heater	125931*
<ul style="list-style-type: none"> • IN USE AND DURING LONG-TERM STORAGE: Every 12 Months • PRIOR TO LONG-TERM STORAGE • POST-MAINTENANCE AND/OR REPAIR 	Functional Testing (Leak/Pressure Testing, Operating Testing & Battery Charging)	See Procedure # 8.3 for Complete Maintenance Steps	Kit, MADM, Service Support	127510

NOTE: For maintenance purposes, procedures listed with a (*) should only be followed for part removal, and not necessarily replacement.

Once removed, certain parts and filters will require cleaning and removal of debris.

A full replacement of the part(s) may not be necessary, unless the parts are damaged, or do not function as intended.

For devices in long-term storage, the internal battery must be recharged annually to maintain battery health.

MADM™ can only be serviced by authorized and qualified personnel in accordance with the recommended periodic maintenance schedule outlined above.

MADM™ Battery Charging Instructions

Follow the below MADM™ Battery Charging Instructions to keep your MADM™ device in optimal working condition.

**THORNHILL
MEDICAL**
Inspiring Innovation

IMMEDIATE ACTION REQUIRED

Immediate Battery Charging Instructions

Upon receiving your MADM™ device:

- Charge the internal battery immediately.
- You will need the MADM™ power plug to complete the charging process.

To Charge the Battery

1. Connect the MADM™ system to wall power using the provided power plug.
2. Allow the battery to charge fully until it displays 100% on the Startup screen.



MADM™ Power Plug



MADM™ Startup Screen



We're Here to Help

Stay connected to MADM™ and the Thornhill Medical Team.



Customer Care is one click away.

Access our Customer Care Portal

<https://thornhillmedical.com/customercare/>



Training and Information in the Palm of your Hand

Download the Training App

<https://thornhillmedical.com/resources/moves-slc/videos/>

+1 416-597-1325 | 1-888-597-1325
60 Wingold Ave., Toronto, ON M6B 1P5 Canada
customer care@thornhillmedical.com | www.thornhillmedical.com