

MOVES[®] SLC[™]

USER MAINTENANCE TRAINING GUIDE



MOVES[®] SLC[™]

USER MAINTENANCE TRAINING GUIDE

Learning Objectives

This Maintenance Training Guide familiarizes users with the requirements needed for maintenance of the MOVES[®] SLC[™] device.

Overview

This User Maintenance Training Guide is a reference tool with a knowledge checklist for maintaining your MOVES[®] SLC[™] device in the field.

The following topics are covered in this training guide:

SECTION ONE: Periodic Maintenance

User Serviceable Components

- Replacing the Ventilator Filter
- Replacing the Hydrocarbon Filter
- Replace the Ventilator Cartridge Liner

Periodic Maintenance Procedures

- System Tests
- System Cleaning
- Exercising the O₂ Concentrator
- Charging the Battery

SECTION TWO: System Test

- Preparing the MOVES[®] SLC[™] for Use
- Inspecting and Installing the Batteries
- Preparing and Connecting AC Power
- Battery Storage & Handling

System Test Instructions

- Steps 1-11

MOVES® SLC™

USER MAINTENANCE TRAINING GUIDE

Getting Started

1. To work through this Maintenance Training Guide you will need to download the following:



[MOVES® SLC™ Training Video](https://www.youtube.com/watch?v=-ZM2nONFKR8)
[Full Version 30 mins](https://www.youtube.com/watch?v=-ZM2nONFKR8)


<https://www.youtube.com/watch?v=-ZM2nONFKR8>



[MOVES® SLC™](https://www.youtube.com/watch?v=-ZM2nONFKR8)
[Maintenance Video](https://www.youtube.com/watch?v=-ZM2nONFKR8)

<https://www.youtube.com/watch?v=-ZM2nONFKR8>

2. The following documents are also available for download on the Customer Care web page: www.thornhillmedical.com/customercare

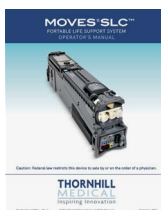

MOVES® SLC™
Operator's Manual



MOVES® SLC™
Quick Start Guide



MOVES® SLC™
Quick Start Guide for
Pre-Use System Tests

The checklist overleaf will guide you through the relevant maintenance and system test areas of the MOVES® SLC™ device.

Once you have reviewed the relevant sections, mark each as complete on the checklist. Review each section as frequently as needed to ensure you are comfortable with the process steps required for each knowledge area.

You're Ready to Go!

MOVES® SLC™

KNOWLEDGE CHECKLIST

MOVES® SLC™ KNOWLEDGE CHECKLIST			
Trainee Name:		Date:	
Section One			
Periodic Maintenance			
	See Operator's Manual Section	View 30 min Training Video	Complete
User Serviceable Components:	16.3.1		
Replacing the Ventilator Filter	16.3.2	@ [22:46]	
Replacing the Hydrocarbon Filter	16.3.3	@ [17:53]	
Replace the Ventilator Cartridge Liner	16.3.4	@ [23:26]	
Periodic Maintenance Procedures:	See Operator's Manual Section	View Maintenance Video	
System Tests	11.2	@ [1:49]	
System Cleaning	16.2	@ [0:22]	
Exercising the O ₂ Concentrator	16.3.5	@ [3:17]	
Charging the Battery	16.3.6	@ [4:38]	

Section Two		
System Test		
Preparing MOVES® SLC™ for Use:	9.14	
Inspect and Installing the Batteries	9.14.1 - 9.14.4	
Preparing and Connecting AC Power	9.14.5 - 9.14.7	
Battery Storage & Handling	9.14.8	
System Test Instructions:	2.1	
Steps 1-11		

MOVES® SLC™

RESOURCES

We're Here to Help

Stay connected to MOVES® SLC™ and the Thornhill Medical Team.

Customer Care is one click away.

Access our Customer Care portal to find important device resources and access the Thornhill Medical team – we're here to help.

<https://thornhillmedical.com/customercare/>



Training and Information in the Palm of your Hand

Download the MOVES® SLC™ training app and have quick access to everything you need, exactly when you need it, no matter where you are.

<https://thornhillmedical.com/resources/moves-slc/videos/>



THORNHILL
MEDICAL
Inspiring Innovation

MOVES[®] SLC[™]

NOTES

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.